

USER MANUAL

PATIENT WASHING TROLLEY





COMPANY INFORMATION

A) MANUFACTURER COMPANY : Gökler Mobilya Medikal Ve Sağlık Hizmetleri Gıda Maddeleri İnşaat Nakliye İmalat Montaj Pazarlama İthalat İhracat Sanayi A.Ş

B) BRAND: NITROCARE

C) ADDRESS: Organize Sanayi Bölgesi 1. Cad. No:7-8 Merkez/SİVAS

D) TEL NUMBER: +90 346 218 19 10-13

E) FAX: +90 346 218 19 14

F) E-MAIL: info@nitrocare.com.tr

G) TYPE: Medical Furnishings and Equipment

H) LIFETIME A PRODUCT: 10 YEARS

Avoiding customer-induced damage and this manual should be read before using the product for best performance.

CONTENTS

1. SAFETY AND HAZARDS.....	1
1.1 Warnings.....	1
1.1.1 Warning Types.....	1
1.1.2 Warning Construction.....	1
1.1.3 Safety Instructions	1
1.1.4 Usage Patient Washing Trolley.....	2
1.1.5 Terms of Use	2
2. STANDARD AND REGULATIONS.....	2
3. OVERVIEW OF PATIENT WASHING TROLLEY.....	3
4. CONTROL ELEMENTS AND FUNCTIONS	4
4.1. Central Lock System Control.....	4
4.2. Protection Bumpers.....	4
4.3. Head Handles.....	4
4.4. Discharge Water Hose.....	5
4.5. Height Pedal.....	5
4.6. Castors	5
5 TECHNICAL SPECIFICATIONS	6
6. CLEANING AND DISINFECTION	6
6.1 Safe <u>Cleanliness</u>	6
7. SPARE PARTS.....	7
8 PROTEC THE ENVIRONMENT.....	7
9 WARRANTY.....	7
10.CONTACT.....	8

1. SAFETY AND HAZARDS

1.1 Warnings

1.1.1 Warning Types

- ✓ Warning ;Ownership indicates possible damage.
- ✓ Warning ; Warning against bodily injury.
- ✓ Hazard;Potential warning against mortal injuries

1.1.2 Warning Construction

	<p>Hazard type and source.</p> <p>Precautions to prevent danger.</p>
--	--

1.1.3 Safety Instructions

- ✓ Carefully follow the directions, a copy of the curriculum is near to the patient's examination table.
- ✓ Replace damaged parts only with original spare parts.
- ✓ If you notice that the patient does not perform the rucksack functions, do not use it and contact the technical service.
- ✓ Give information about the use functions for the patient and other users.
- ✓ Safe use weight should never be exceeded. (150kg)
- ✓ Only spare parts supplied by authorized servicing must be used. When the spare parts are supplied, the manufacturer shall not be responsible for any damage, damage or injury.
- ✓ The patient washing trolley should be wiped with a damp cloth by cleansing and disinfection solution at room temperature.

1.1.4 USAGE PATIENT WASHING TROLLEY

The patient was designed to for wash the patient in a safe way.

 A yellow triangular warning sign with a black exclamation mark inside.	✓ Safe use weight should never be exceeded. (150kg)
--	---

1.1.5 Terms Of Use

Conditions of use must be met. should be used under the below environmental conditions.

- temperature range-10° C Or 40° C must be among
- Relativehumidity % 30 and % 75 must be among

STANDARD AND REGULATIONS

2. STANDARD AND REGULATIONS

Conforms to the below standards and regulationson the Patient Washing Trolley .

- 93/42/EEC
- The below quality management system is applied.
-
- TSE EN ISO 9001
- TS EN ISO 14001
- TS EN ISO 18001
- TS EN ISO 13485
- TS EN ISO 17025

3. OVERVIEW OF PATIENT WASHING TROLLEY



1 Central LockSystem Control.

2 ProtectionBumper.

3 HeadHandles.

4 WaterDischargeHose.

5 Height Pedal.

6 Castors.

4 CONTROL ELEMENTS AND FUNCTIONS

4.1. Central Lock System Control



- The brakes are switched off when the brake on pedal press.
- The brake pedal is turned on when lifted up.
- The brake system is used as central brake.

4.2. Protection Bumper



- Bumper wheels ensures that the carrier is protected against impact.

4.3. Head Handles



- Patient washing trolley to give an angle. When the hand is depressed, the shock absorbers become active and the top of the stretcher is angled.

4.4. Water Discharge Hose



- The patient is designed to for the drain water during washing.

4.5. Height Pedal



- The pedal designed for the stretcher move up and down.
- Pedal being functions by pumping.

4.6. Castors

- Wheels are locked centrally and It a special insulated wheel from the wheels.



Be careful that the brakes are closed during patient washing.

5 TECHNICAL SPECIFICATIONS

PATIENT WASHING TROLLEY TECHNICAL DIMENSIONS	
MOVEMENT	CAN MAKE HEIGHT MOVEMENT WITH HYDRAULIC LIFT.
MECHANICAL PROPERTY	IT CAN MAKE TRENDELENBURG MOVEMENTS WITH THE HELP OF TWO ABSORBERS.
RAILING STRUCTURA AND TYPE	THE SIDE RAILS ARE MADE OF STAINLESS METAL MATERIAL. (304 QUALITY)
HYDROULIC PROPERTIES	HEIGHT LIFTING FORCE 4500 N (1 PIECES)

6. CLEANING AND DISINFECTION

Metal surfaces are coated with electrostatic powder paint and stainless. As a result, a moist dust can easily be cleaned with a cloth or sponge.

6.1. SafeCleanliness

- Do not use abrasive powders, steel wool, or other material and cleaning materials that may cause damage
- Do not use solvents and detergents that may affect the plastic structure and consistency (benzene, toluene, acetone, etc.)
- Clean thoroughly with a damp cloth.
- Do not use pressure cleaners.

4. SPARE PARTS

- ✓ Defects caused by usage errors are outside the warranty scope. Can not be requested spare parts.
- ✓ The care should be done by a trained hospital staff. If any problem is encountered, please inform our technical service unit

Request for spare parts and information

- ✓ NITROCARE TECHNICAL SERVICE
- ✓ servis@nitrocare.com.tr
- ✓ 0542 845 23 05

5. PROTECT THE ENVIRONMENT

NITROCARE is the idea of taking great steps in the protection of the environment by thinking about future generations.

This product's materials are environmentally compatible. Does not contain dangerous substances.

Customer is informed with used symbols and signs.

The product packaging is made with recycled boxes.



6. WARRANTY

NITROCARE will be only be held liable for service and product reliability on a regular basis.

If a serious irreparable problem crop up during maintenance !

- Do not continue to use the patient washing stretcher!
- This product is under warranty for 24 months from the date of purchase.
- Errors caused by production and assembly mistakes are made free of repair.
- Defects resulting from the use of the unit are not extent by the warranty.

7. CONTACT

FACTORY/ SİVAS

Organize Sanayi Bölgesi 3.Kısım 5.Cadde No:48 – 50 SİVAS-TURKEY

Phone: +90 0 346 218 12 91 – 92 – 93

Fax: +90 0 346 218 12 95

E-Mail: info@ofisline.com.tr

FACTORY/ SİVAS

Organize Sanayi Bölgesi 1.Cadde No:7 SİVAS-TURKEY

Phone: +90 346 218 19 10 – 11 – 12 – 13

Fax: +90 346 218 19 14

E-Mail: info@nitrocare.com.tr

CENTER/ ANKARA

Kızılırmak Mah.Muhsin Yazıcıoğlu Cad.1436 sok No:1-37 Çukuranbar/ANKARA-TURKEY

Phone:+90 312 286 66 44

Fax:+90 312 286 86 67

E-Mail:info@nitrocare.com.tr

SALES OFFICE/ İSTANBUL

Barbaros Mah.Halk Cad No:15-17 Ataşehir-İSTANBUL TURKEY

Phone:+90 216 317 56 50-51

Fax:+90 216 317 56 62

E-Mail:info@nitrocare.com.tr

SALES OFFICE/ İZMİR

9 Eylül Mah.Akçay Cad.No:159 Gaziemir İZMİR-TURKEY

Phone:+90 232 220 13 13-14

Fax:+90 232 220 13 15

E-Mail:info@nitrocare.com.tr



OFISLINE	NITROCARE	dekoline	ZARONI	veOfis.com	profisimölsun	house
----------	-----------	----------	--------	------------	---------------	-------