

USER MANUEL

MANUAL PEDIATRIC BED



NH 1110 S





Authorized Link : info@nitrocare.com

Phone : 0346 218 19 10 - 13

Fax : 0346 218 19 14

Address: Organize Sanayi Bölgesi 1. cad. No:7 Merkez / SİVAS-TURKEY

Gökler Mobilya Medikal ve Sağlık Hizmetleri Gıda
Maddeleri İnşaat Nakliye İmalat Montaj Pazarlama İthalat İhracat Sanayi Ticaret A.Ş.

NITRO HB models are designed essentially with maximum safety and mobility. They are designed considering the physical disability and comfort for the highly risked patients.

This user manual must be read clearly through to avoid customer related damages and to achieve the best performance of the product.

INDEX

1. SECURITY AND DANGERS	1
1.1 Warnings	1
1.1.1 Warning Type	1
1.1.2 Warning Structure	1
1.1.3 Safety Instructions	1
1.1.4 Pediatric Bed Use	2
1.1.5 Terms of Use	2
2. STANDARDS AND REGULATIONS	2
3. OVERVIEW OF PEDIATRIC BED	3
4. CONTROL ELEMENTS AND FUNCTIONS	4
4.1. Side Guardrail On / Off Button	4
5. TECHNICAL SPECIFICATIONS	5
6. CLEANING, DISINFECTION	6
6.1. Safe Cleaning	6
7. SPARE PARTS	6
8. ENVIRONMENTAL PROTECTION	6
9. WARRANTY	7
10. CONTACT	7

1. SECURITY AND DANGERS

1.1 Warnings

1.1.1 Warning Type

- ✓ Caution indicates possible damage to property.
- ✓ Warning Warning against personal injury
- ✓ Danger potential warning against fatal injuries.

1.1.2 Warning Structure

	Hazard type and source
	Precautions to avoid danger

1.1.3 Safety Instructions

- ✓ Carefully follow the instructions, if necessary, keep a copy of the instruction near the patient's examination table.
- ✓ Only the original spare parts and damaged parts should be replaced immediately and the parts should be replaced by qualified personnel.
- ✓ If it is noticed that it does not perform the functions of the bed, do not use it and inform the technical service.
- ✓ Provide information to patients and other users about usage functions.
- ✓ Safe usage weight should never be exceeded. (150kg)
- ✓ Only spare parts supplied from the authorized service should be used. When using spare parts from other suppliers, the manufacturer does not accept any responsibility for any damage, damage or injury.
- ✓ It should be wiped with a damp cloth with water cleaning and disinfection solution at room temperature.

1.1.4 Pediatric Bed Use



Safe working load should not be applied over 150 kg.

1.1.5 Terms of Use

The conditions of use must be met and used under the ambient conditions stated below.

- Temperature Range must be between -10 ° C and 40 ° C.
- Relative humidity is between 30% and 75%.

2. STANDARDS AND REGULATIONS

- The patient examination table complies with the standards and regulations below.
- 93/42/EEC

The quality management system below is implemented.


- TSE EN ISO 9001
- TS EN ISO 14001
- TS EN ISO 18001
- TS EN ISO 13485
- TS EN ISO 17025

3. OVERVIEW OF PEDIATRIC BED



4. CONTROL ELEMENTS AND FUNCTIONS

4.1. Side Rail Open - Close

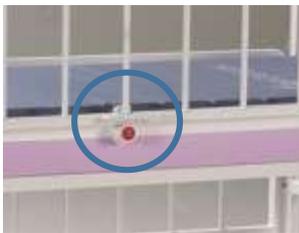


- When the button located on the lower side of the side railing is pressed, the side railing moves down.



Side railing should be supported with the other hand in the side railing down movement.

5. TECHNICAL SPECIFICATIONS

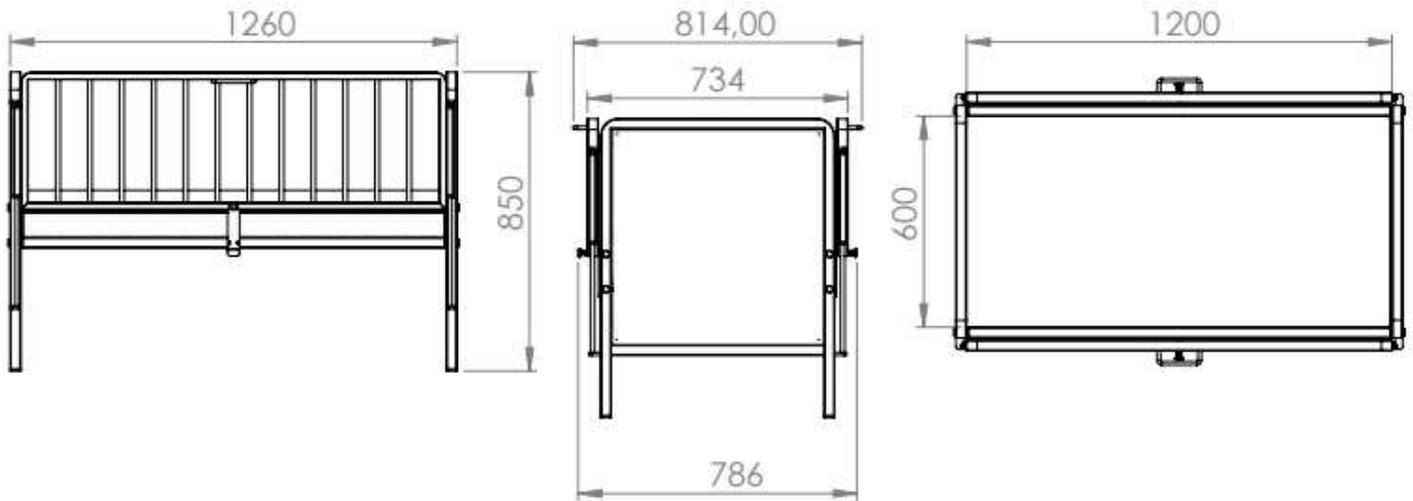

6. CLEANING, DISINFECTION

Metal surfaces are covered with electrostatic powder paint. Accordingly, it can be easily cleaned with a damp dust cloth or sponge.

6.1. Safe Cleaning

- Do not use abrasive powders, steel wool, or other material that could damage it.
- Do not use solvents and detergents (benzene, toluene, acetone, etc.) that can affect the plastic structure and consistency.
- Clean with a well-wrung damp cloth.
- Do not use pressure cleaners.

7. SPARE PARTS

- Faults caused by usage errors and spare parts demand are made out of warranty.
- The maintenance should be done by trained hospital staff and if any problems are encountered, the technical service unit of our company should be informed.

Spare parts requests and information received

- ✓ NITROCARE teknik servis
- ✓ Teknik servis@nitrocare.com.tr
- ✓ 0542 845 23 05

8. ENVIRONMENTAL PROTECTION

NITROCARE carries the thought of taking big steps in protecting the environment by considering the next generations.

The materials of this product are environmentally compatible. It does not contain dangerous substances.

The customer is informed with the symbols and signs used.

Product packaging is made with recycled boxes.



9. WARRANTY

NITROCARE will only be held responsible for service and product reliability on a regular basis.

If a serious irreparable problem occurs during maintenance.

Do not continue using the patient washing stretcher!

This product is under warranty for 24 months from the date of purchase. Errors arising from production and assembly errors are made free of charge. Malfunctions caused by the use error are not covered by the warranty.

10. CONTACT

FABRİKA / SİVAS

Organize Sanayi Bölgesi 3.Kısım 5.Cadde No:48 – 50 SİVAS-TÜRKİYE

Telefon: +90 0 346 218 12 91 – 92 – 93

Faks: +90 0 346 218 12 95

E-Mail: info@ofisline.com.tr

FABRİKA / SİVAS

Organize Sanayi Bölgesi 1.Cadde No:7 SİVAS-TÜRKİYE

Telefon: +90 346 218 19 10 – 11 – 12 – 13

Faks: +90 346 218 19 14

E-Mail: info@nitrocare.com.tr

MERKEZ / ANKARA

Kızılırmak Mah. Muhsin Yazıcıoğlu Cad.1436 sok No:1-37 Çukuranbar/ANKARA-TÜRKİYE

Telefon:+90 312 286 66 44

Faks:+90 312 286 86 67

E-Mail:info@nitrocare.com.tr

SATIŞ OFİSİ / İSTANBUL

Barbaros Mah. Halk Cad No:15-17 Ataşehir-İSTANBUL TÜRKİYE

Telefon:+90 216 317 56 50-51

Faks:+90 216 317 56 62

E-Mail:info@nitrocare.com.tr

SATIŞ OFİSİ / İZMİR

9 Eylül Mah. Akçay Cad.No:159 Gaziemir İZMİR-TÜRKİYE

Telefon:+90 232 220 13 13-14

Faks:+90 232 220 13 15

E-Mail:info@nitrocare.com.tr

